



Prophecy[®] Gap Balancing

Envision the Results[™]

Product Specification Sheet



MicroPort

Orthopedics

Integrity In Motion[™]

Getting Started

The PROPHECY® Case Management Website has been created to allow surgeons to:

- Upload CT/ MRI Scans*
- Review Cases
- Check case status
- Approve pre-operative plans

* NOTE:

For appropriate scan instructions, please refer to the CT Scan Protocol or MRI Scan Protocol included in the PROPHECY® Gap Balancing Surgical Technique

To gain access to the PROPHECY® Case Management Website, a request must be sent by the sales rep. to prophecy@ortho.microport.com.

This applies to both existing and new users.

This request needs to specify for which product the access is needed:

- aMP™ Standard Prophecy Guides
- aMP™ Balancing Prophecy Guides
- eMP™ Standard Prophecy Guides
- eMP™ Balancing Prophecy Guides

IMPORTANT:

The PROPHECY® Gap Balancing Guides and PROPHECY® Standard Pre-Operative Navigation Guides are not interchangeable.

Additionally, the PROPHECY® Gap Balancing technique requires a system-specific (aMP™ or eMP™) reusable instrument kit. When the surgeon starts planning his cases and the Balancing option is selected, a warning will appear, enforcing the need for an instrument kit.

MicroPort Orthopedics Create Motion® PROPHECY™ Pre-Operative Navigation Guides Simple. Fast. Accurate. Envision the Results.™ covered by PCA, Inc. browser requirements

Welcome, Sales

Surgeons Cases (6) Calendar Profile Contact Us Log Out

Current Cases

Add New Case

View Past Cases

Product Documentation

ADVANCE (2) ADVANCE Gap Balancing (0) EVOLUTION (9) EVOLUTION Gap Balancing (0) All (11)

Edit	Report	Case #	Side	Status	Surgeon	Patient	Surgery
No data to display							

Patient Specific Guides

TIBIAL GUIDE

Features the same built in resection block or pin alignment options available in the traditional PROPHECY® system.



Tibial Pin Alignment Guide


Anterior holes align pins of Tibial Resection Guide to control resection depth, varus/ valgus alignment and posterior slope.

Tibial Pin Alignment and Resection Guide

Accepts Tibial Resection Guide.

FEMORAL GUIDE

- Designed for flexion and extension balancing and varus/valgus and rotational alignment.
- Final pinning for varus/ valgus and rotational alignment are made through the femoral guides utilizing reusable, embedded pinning guides.

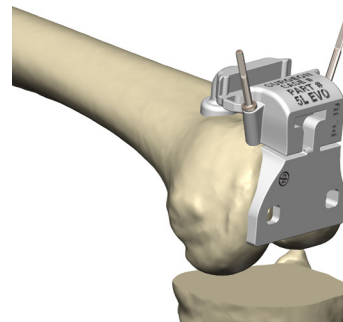

POSTERIOR FLEXION BLOCK

- Single-use, patient-specific block that mimics the final implant thickness past full extension to 90 degrees of flexion to assess the flexion gap. The posterior flexion block contacts with the posterior-most aspect of the femur guide and extends around the posterior condyles of the femur.



Notes

Lined area for writing notes, consisting of 20 horizontal lines.

Ordering Information

	aMP™	eMP™
Prophecy® Balancing Instrument Kit	PBADVKT1	PBEVOKT1
Prophecy® Balancing Surgical Technique	011079	011079
Medial Pivot Surgical Technique	008016	009787
Prophecy® Balancing Design Rationale	011127	011127
Sales Leave-Behind	011126	011126

Bill of Materials

PBADVKT1

Item No.	Description	Quantity	UM
PBSP2000	Prophecy® 2MM Spacer	1	EA
PBSP4000	Prophecy® 4MM Spacer	2	EA
PBDEG100	Prophecy® 1 DG Wedge	1	EA
PBDEG200	Prophecy® 2 DG Wedge	1	EA
PBDEG300	Prophecy® 3 DG Wedge	1	EA
PBDEG400	Prophecy® 4 DG Wedge	1	EA
PBAPG410	Prophecy® Advance® 4-IN-1 Pin Guide	1	EA
PBAPG400	Prophecy® Advance® Distal Pin Guide	1	EA
PBAGF400	Prophecy® Advance® Guide Frame	1	EA
E2001020	Evolution® Tibial Base Trial Handle	1	EA
KSPBATR1	Prophecy® Balancing Instrument Tray	1	EA

Bill of Materials

PBEVOKT1

Item No.	Description	Quantity	UM
PBSP2000	Prophecy® 2MM Spacer	1	EA
PBSP4000	Prophecy® 4MM Spacer	2	EA
PBDEG100	Prophecy® 1 DG Wedge	1	EA
PBDEG200	Prophecy® 2 DG Wedge	1	EA
PBDEG300	Prophecy® 3 DG Wedge	1	EA
PBDEG400	Prophecy® 4 DG Wedge	1	EA
PBEPG500	Prophecy® Evolution® Pin Guide	2	EA
PBEGF500	Prophecy® Evolution® Guide Frame	1	EA
KSPBATR1	Prophecy® Balancing Instrument Tray	1	EA



MicroPort Orthopedics Inc.
5677 Airline Road
Arlington, TN USA 38002
866.872.0211

MicroPort Orthopedics BV
Hoogoorddreef 5
1101 BA Amsterdam
The Netherlands

ortho.microport.com

Trademarks and Registered marks of Microport Orthopedics Inc.
© 2015 Microport Orthopedics Inc. All Rights Reserved. 011160